


**STEP 1:**

**Transfer** the desired amount of compounded cream into the UnoDose™ Applicator barrel using a spatula, ointment jar, or filling system.


**STEP 2:**

For best results, remove air during filling. Start with a small amount to **fill the corners** in the bottom of the barrel. **Tap the bottom** of the applicator on your palm or a countertop to settle the cream and remove air.


**STEP 3:**

**Add the remaining cream** in several steps while continuing to **tap the bottom** of the applicator to settle the cream and remove air. Leave the top of the cream flat and smooth.


**STEP 4:**

Put the applicator cap on the barrel and **push downward** to snap into place. The applicator cap will butt up against the ring on the barrel. **Tap the bottom** again and confirm that no air remains in the cream.


**STEP 5:**

**Prime to remove air.** For a fast prime, **use the prime tool supplied in each box.** **Push slowly** until cream just comes out the holes, or the tool stops.

**NOTE:** The short end of the tool leaves 32 usable millimeters in the applicator, the long end leaves 17.


**STEP 6:**

Finish priming by **turning the dosing knob** several clicks until cream exits all holes smoothly and equally.

**NOTE:** Be sure to leave the knob in a "lock" position, and not between clicks.


**STEP 7:**

**Wipe away the cream** from the applicator cap, then **lock the protective cover** in place with a light push and feel a click.


**STEP 8:**

Confirm the correct volume on the scale by **reading at the blue line.**

**NOTE:** The scale shows the usable volume and clicks remaining, and excludes the residual volume of 1.5 millimeters.