

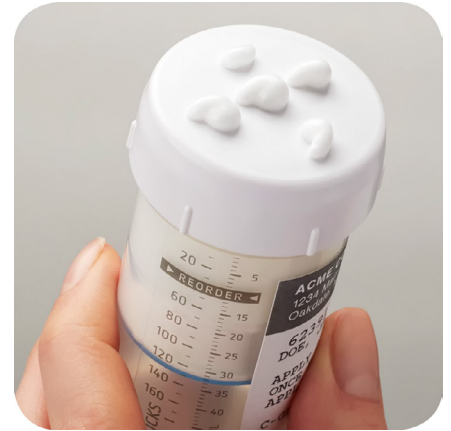

**STEP 1:**

**Remove** the protective cover with a light counterclockwise twist and pull while holding the barrel.


**STEP 2:**

Turn the dosing knob in the direction marked for the number of prescribed clicks.

***NOTE:** Be sure to leave the knob in a "lock" position, and not between click positions.*


**STEP 3:**

**Confirm** that cream has exited the holes of the applicator cap.


**STEP 4:**

**Apply** the cream to the skin absorption site. **Rub** with the applicator cap until the cream is evenly spread (per physician or pharmacist direction). **Replace** the protective.


**STEP 5:**

**Check** the scale at the blue line for the number of clicks remaining. **Refill** your prescription as instructed by your pharmacy.