



# MD<sup>®</sup> SYRINGE / INSTRUCTIONS FOR USER



## STEP 1:

Turn the base of the MD Syringe counter clockwise into the OPEN position. Make sure to hear the click.



## STEP 2:

Press the base to dispense on localized area.



## STEP 3:

Apply the preparation using the soft silicone tip by rubbing it into the area of application as directed.



## STEP 4:

Turn the base of the MD Syringe clockwise into the CLOSE position. Make sure to hear the click.



## STEP 5:

Call for replacement, once the piston nears the domed part of the view window.

**CONTACT YOUR PROVIDER FOR MORE INFORMATION**