



MEDISCA[®]
YOUR TRUSTED PARTNER IN COMPOUNDING

PRODUCT NO.: 7823







MEDISCA ADAPTER CAP (F, 28mm)

28-415, LDPE, Green (Long Neck)

DESCRIPTION: Match the bottle closure-code with the MEDISCA Adapter Cap of your choice for easy connection to bottle tops for simplified oral syringe filling. Please contact a MEDISCA Sales representative for your very own MEDISCA Adaptor-Chek, for quick identification of bottle threads.

FEATURES:

-  Made out of low-density polyethylene (LDPE) with a tethered, re-closable cap
-  Non-DEHP
-  Used on most glass or plastic bottles to draw liquid
-  Latex-free


ADDITIONAL INFORMATION:

Closure Code:

| Letter | Closure Code | mm Code | GPI/SPI Neck Finish |
|---------------|---------------------|----------------|----------------------------|
| A | 18-400 | 18 | 400 |
| B | 20-400 | 20 | 400 |
| C | 22-415 | 22 | 415 |
| D | 24-410 | 24 | 410 |
| M | 24-415 | 24 | 415 |
| E | 28-400 | 28 | 400 |
| K | 28-410 | 28 | 410 |
| F | 28-415 | 28 | 415 |

Closure Code Explanation:

- 1) The closure code consists of the cap diameter (mm) and neck finish.
- 2) The neck finishes that are the most common (and that we will work with) are 400, 410, and 415. If you match a bottle neck below with the picture, you will be able to tell the finish.

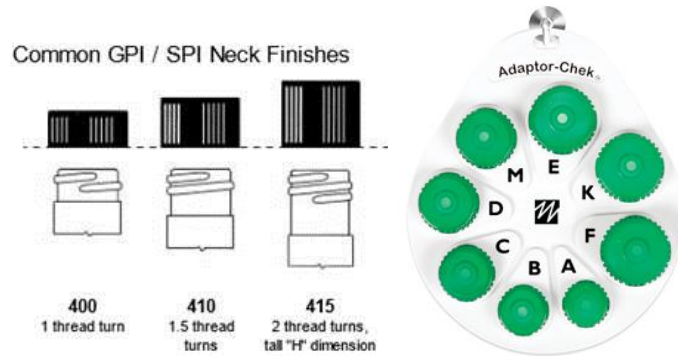

- 3) The closure codes must match, e.g., a 24-410 bottle will only fit a 24-410 cap (adapter-cap D).
- 4) You will see that some Adapter-caps share the same diameter and thus can be easily confused even when using the Adaptor-Chek (seen above). Problems will arise if a D adapter-cap is being used instead of an M, or the like.