



MEDISCA[®]
YOUR TRUSTED PARTNER IN COMPOUNDING

PRODUCT NO.: 8412



TOPI-CLICK APPLICATOR

35mL, 0.5mL per 2 Clicks, Plastic, Purple

DESCRIPTION: Topi-Click is a metered cream/gel dispenser and applicator for topical compounds and is easy to use for all types of patients. Topi-Click is easier to use and fill than oral dosing syringe.

FEATURES:




- ✔ Domed applicator pad for hands-free application – hand absorption can affect treatment outcome by reducing dose absorbed by prescribed treatment area
- ✔ Audible, visual and tactile click makes it easy for all patients to accurately dose medications
- ✔ “Refill” indication for prompting on-time re-buys that help eliminate missed applications
- ✔ Lower cost with simplified labeling – labels as fast and as easily as a prescription bottle
- ✔ Easier to fill and use with increased control over plunger movement than a topical syringe
- ✔ Now equipped with a flip tab for easier leverage to pop the top
- ✔ Friendly reminders of instructions on turn-dial base to help patients
- ✔ Applicable for total dispense weights of approximately 5g to 35g of topical preparations
- ✔ Turn the base “1 click” (a ¼ turn) to dispense one metered amount; doubling or tripling the dose amount is as simple as turning to the 2nd or 3rd click
- ✔ Made of FDA medical-grade polypropylene material
- ✔ UV block plastic protects the contents from UV rays

ADDITIONAL INFORMATION:

Applicable Products:

- ✔ 7947: Topi-Click Filling Station
- ✔ 7785: Modified 1L Unguator Jar
- ✔ 7778: Topi-Click Filling Nozzles

User Instructions:

| | |
|---|--|
| <p>Step 1</p>  <p>Pop off the cap of the Topi-Click® container by placing your thumb under the tab on the protective outer cap and pressing upward.</p> | <p>Step 2</p>  <p>Turn the base to dispense the number of clicks specified on label.</p>  <p>When rotating base, watch the Visual Indicators</p> <p>Indicator Node on Body is fixed at the Click Point</p> <p>Indicator Column moves with Base. This shows Node in line with column.</p> <p>LEAVE IN THIS POSITION AFTER EVERY CLICK (The Home Position)</p> |
|---|--|

Step 3




Apply treatment by rubbing the cream, which is now on the top of the applicator pad, to the area of skin as directed.

Step 4



Prevent evaporation of next dose, by snapping the cap on completely.

Step 5



The **bottom** of the plunger is the indicating line; when it reaches the embossed line **above** the word "Refill," there are approximately 12 clicks left. Approx. 32 clicks remain at the line **below** the word "Refill" as shown above.